

AT51111

Lightweight Wheeled Walker



CE

1. INTRODUCTORY REMARK

We are thankful that you decided on the *AT51111* lightweight wheeled walker.

The quality of the wheeled walker will not disappoint you.

Please read through the instruction manual thoroughly before using the wheeled walker for the first time.

It contains important safety information and valuable tips for the proper use and maintenance of the wheeled walker. If you have questions or need additional information, refer to your specialized dealer who delivered *AT51111* your wheeled walker. They will be happy to give you advice.

The instruction manual is also available for download and can be used in your desired format, even for people with vision problems.

2. TABLE OF CONTENTS

1. INTRODUCTORY REMARK	2
2. TABLE OF CONTENTS	3
3. GENERAL INFORMATION	5
Specific Function	5
Indications	5
Contraindications.....	5
Duration of Use.....	6
Liability	6
Declaration of Conformity.....	6
Symbols	6
4. PRODUCT AND DELIVERY SUMMARY	7
Delivery Inspection	7
Scope of Delivery	7
Identification	7
Overview of CLASSIC Pocket.....	8
5. CLASSIC POCKET Setup	9
Handle Height.....	9
Pouch.....	9
Brakes.....	10
Increase/lessen brake effect	10
6. USE OF CLASSIC POCKET	11
Opening and Folding Up.....	11
Transporting and Storage.....	12
Running with the Wheeled Walker.....	13
Tilting Aid.....	13
Brakes and Parking Brake	14
Wheeled Walker as a Seat.....	14
Lighting.....	Błąd! Nie zdefiniowano zakładki.
Carrying the Wheeled Walker	15
Safety Information	15
7. CLEANING AND DISINFECTION	15

Cleaning.....	15
Disinfection.....	16
8. RE-USE.....	16
9. DISPOSAL.....	16
10. TECHNICAL DATA	17
11. REPAIR AND MAINTENANCE FOR THE SPECIALIZED SHOP	18
12. WARRANTY	19

3. GENERAL INFORMATION

The instruction manual at hand will help you to be more familiar with the operation and handling of the wheeled walker. This instruction manual is a fixed and necessary component of the wheeled walker. Keep this instruction manual handy and give it to the next person if passing the wheeled walker over to someone else. Please read the instruction manual through completely prior to the first use of the wheeled walker.

Specific Function

The wheeled walker *AT51111* is designed for indoor and outdoor spaces. It is designed exclusively as a walker with seating for one person. Carrying of more than one person is not allowed. If not otherwise described, this instruction manual is directed to the patients as the operator.

Indications

The wheeled walker is suited for people with limited mobility who still have sufficient ability, strength and stability to be able to leave their residence with support of the wheeled walker.

Contraindications

Use of the wheeled walker is not suitable with

- Perception issues
- Severe equilibrium issues
- Limb loss of the arms
- Inability to sit

Duration of Use

The expected life span of the wheeled walker is up to 5 years with use, safety, care and maintenance in accordance with regulations. Beyond this time period, the wheeled walker can still be used as long as still in proper condition.

Liability

As the manufacturer, we can only be considered liable for the safety, dependability and usability of the wheeled walker if modifications, upgrades, repairs and maintenance work are completed only by certain people who are authorized by us and the wheeled walker is used allowing for all handling information.

Declaration of Conformity

Medical Medizin- und Rehaprodukte GmbH [a limited liability company] declares as manufacturer of the product that the wheeled walker *CLASSIC Pocket* conforms to the requirements of the European Guidelines 93/42/EEC.

Symbols



Important! Particular safety-related information.
Non-observance can lead to injury or defects on the product.



Information! Particularly useful information.



Follow instruction manual



CE marking

4. PRODUCT AND DELIVERY SUMMARY

Delivery Inspection

The AT51111 wheeled walker is delivered completely assembled in a box. After unpacking, retain the box and the packaging material if possible. They can be used for subsequent possible storage of the wheeled walker.

Inspect the delivered contents upon receipt for completeness and sound condition. In the case of damages, do not use the product yet. First notify the dealer.

Scope of Delivery

Contents consist of the following main components:

- 1 outer packaging
- Wheeled walker including back belt and tilt support
- Mesh pocket with reflector and zipper
- 2 cane holders with inner shell
- Instruction manual
- 1 LED position lamp

Optionally included:

- Large transporting case
- Tray

Identification

Each wheeled walker is furnished with a label. This is located on the interior of the left frame. Please produce the serial number on the warranty section.

Figure 1: Label

Overview of AT51111



Figure 2: Product overview

- 1 Seat back
- 2 Pouch
- 3 Front wheels mounting
- 4 Front wheels (steerable)
- 5 Brake adjusting screw
- 6 Rear wheels
- 7 Tilt support (left and right)
- 8 Rear wheels mounting
- 9 Lighting
(optional)
- 10 Sitting surface
- 11 Sliding height adjustment handle
- 12 Brake handle
- 13 Sliding handle

5. AT51111 Setup

We recommend you setup the wheeled walker together with your specialized dealer. With enough manual skills, you can also complete this by yourself.



Be aware of the increased risk of entrapment during setup of the wheeled walker.

Handle Height

You can fit the height of the handle to your body size. This does not require any tools.

A red latch (Figure 3) is found beneath every handle. Push the latch in and keep it pressed. Now you can raise or lower this handle. Let go of the button. Pull or push yet again until you hear and notice that the handle clicks into place and can no longer be adjusted. Repeat the process on the other side until both handles are set to the same height.



Figure 3

Pouch

The wheeled walker is delivered with an assembled pouch. The pouch can simply be taken off or hung above the hanger (Figure 4).



Figure 4

Brakes



Setup of the brakes should be completed by a trained specialist. However, if you have sufficient experience, you can complete this job by yourself. Keep in mind though that incorrectly installed brakes can have life-threatening consequences.

Increase/lessen brake effect

1. Loosen Allen screw (Figure 5, arrow 1).
2. Retighten or untighten Bowden cable by pulling out or pushing in (Figure 5, arrow 2).
3. Refasten Allen screw.
4. Small adjustments:
Loosen counter nut (Figure 6, arrow 1) so that the setup screw (Figure 6, arrow 2) can be turned.

Turn the setup screw clockwise: brake effect lessens. Turn the setup screw counterclockwise: brake effect increases.

Finally, retighten the counter nut.

Complete this step again with the other brake.

Adjust the brakes so that when moving it does not rub and it is easy to operate. With actuator brakes, you are not able to move the walker.

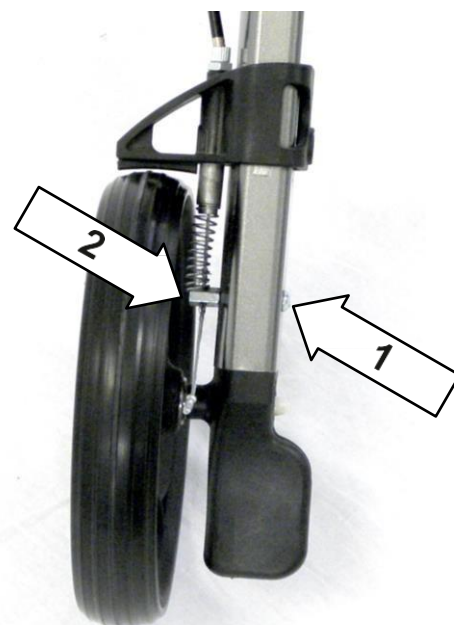


Figure 5

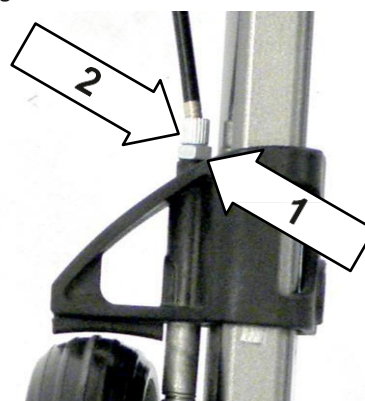


Figure 6

6. USE OF AT51111



Be aware of risk of entrapment, especially during opening and folding up as well as when preparing to transport

Opening and Folding Up

The wheeled walker can be stored in a folded up position for space savings.

In order to be able to use it, open it up by pushing on the seat from the outer edges (Figure 7). On the bottom side of the wheeled walker find the locking device in order to prevent unintended folding of the walker. When opening the walker up a certain click is heard, the locking device is properly engaged. To fold, pull handle up on the seat (Figure 8).



Figure 7



Figure 8

Transporting and Storage

In order to be able to transport and store the wheeled walker for space savings, it is possible to pivot the front wheels up and take off the mounting bracket of the rear wheels (Figure 9).



Figure 9

Front Wheels

Press the red latch (Figure 10) in order to loosen the lock on the front wheels. Then pull the front wheels forward and turn 180° upwards.

In order to use the wheeled walker again, pull the front wheel forwards and pivot it downwards.

It audibly clicks in place; control the fixed seat.



Figure 10

Rear Wheels

Next, fold down the lever (Figure 11). Finally, press the push-button and pull out the rear wheel assembly downwards. Be aware that the rear wheel is still always connected to the wheeled walker by the brake cable.

To assemble, simple push the rear wheel assembly back into the frame. It audibly clicks in place. Now fold the lever back.

Make sure that the rear wheel is securely fixed and the brakes function correctly.

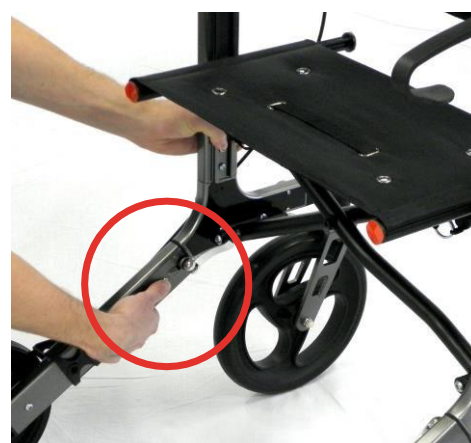


Figure 11

Running with the Wheeled Walker

First test the use of the wheeled walker on a level, dry surface. Allow yourself to be accompanied by another person if possible. Practice after the first attempt – if you feel safe – using it on sloped surfaces.



Pay attention when going with the wheeled walker to remain close to the sliding handles. Only in doing so, can the wheeled walker effectively reduce the risk of falling. Eliminate a slouched back.

You determine the speed and direction by pushing the wheeled walker. Use both brakes in order to slow down your speed. Pay attention when braking to always engage both brakes. The wheeled walker will otherwise turn with the unbraked wheel and you can fall.

Attempt to go around obstacles in your path. With steps, look for a ramp; with stairs, take the elevator. If this is not possible, the wheeled walker has to be carried over the obstacle. When possible, try to find another person to help you.

Tilting Aid

Your wheeled walker is equipped with two tilting aids (Figure 12), which should ease getting over curbs for you. As you step on the tilting aid with your foot and pull back the on the handles at the same time, raise the front wheels. In this manner, you can get over curbs and other small obstacles with little effort.

Please pay attention when doing so for a fixed condition.



Figure 12

Brakes and Parking Brake

Your wheeled walker is equipped with a charged operating brake and with a parking brake to prevent accidental rolling away.

In order to easily and safely bring the wheeled walker to a stop, pull both brake handles upwards (Figure 13).

The brake effect depends on how hard the brakes are pressed. Use the parking brake by pushing the brake handles downwards until they lock into place with an audible click. (Figure 14).

To loosen the parking brake, pull the brake handles back up. (Figure 15).

Wheeled Walker as a Seat

Your wheeled walker is equipped with a sitting surface and a seat back. Before sitting, you absolutely must put on the parking brake. The wheeled walker can otherwise roll away and you could fall. Do not use any additional seat cushions; it could slip and you could fall.



Figure 13



Figure 14



Figure 15

Carrying the Wheeled Walker

If you would like to carry the wheeled walker, over a curb for example, fold it together. Carry the wheeled walker with both hand grips or by the frame. Be aware of the risk of entrapment.

Safety Information

Prior to each use, the wheeled walker must be examined for damages and the brakes must be inspected for proper function.

The wheeled walker is not suitable for use on escalators (risk of falling!).

The wheeled walker's material is not flame-retardant.

The maximum operator weight is 136 kg [300 pounds]; the maximum load capacity of the carrying pouch is 5 kg [11 pounds]. The wheeled walker cannot be used for the transportation of loads. The carrying pouch is not suitable for transporting animals.

The wheeled walker can be folded together for space savings. Please be aware that there is risk of entrapment while folding up and opening. Do not pull on attached pieces (brakes, wheels, basket, etc.) while folding or unfolding the wheeled walker, but rather always on the walker's frame.

Please be aware that cushion parts that are placed directly in the sunlight heat up and could cause injury to the skin upon contact. Therefore, cover these parts or protect the wheeled walker from solar radiation

7. CLEANING AND DISINFECTION

Cleaning

The frame pieces can be wiped with a damp cloth. For heavy dirt, a mild cleaning agent can be used.

The wheels can be cleaned with a damp brush with plastic bristles (do not use wire brushes!)

The seat back, seat and transporting pouch can be washed with a mild soap.



Do not use a high pressure or steam cleaner!

Dry the wheeled walker afterwards with a towel or let it air dry.

Disinfection

The wheeled walker can be disinfected with a common household disinfectant. Use an approved alcohol based spray disinfectant or wipe disinfectant. The Robert Koch list of approved disinfectants can be found at <http://www.rki.de>.

Allow the wheeled walker to air dry and be aware manufacturer's treatment time for the disinfectant.

8. RE-USE

The *AT51111* wheeled walker is suitable for re-use. The wheeled walker is to be initially cleaned and disinfected according the attached care and hygiene information. During the transfer of the wheeled walker, think about handing over all necessary technical documents to the new user for safe handling. The wheeled walker must be inspected by an authorized specialist and be transferred in proper condition.

9. DISPOSAL



When your wheeled walker will no longer be used and should be disposed, please contact your specialized dealer. If you would like to undertake disposal for yourself, ask the local waste management company for the disposal guidelines of your residence.

10. TECHNICAL DATA

Size and Weight	
Seat width	450 mm
Seat height	540 mm
Total width	653 mm
Total length	706 mm
Total height/ Height of handle (adjustable at 6 levels)	810 – 936 mm
Empty weight (without accessories)	7,85 kg
Maximum load capacity	136 kg
Maximum load capacity of mesh pouch	5 kg
Folding size (cm)	706 x 250 x 810 mm (L x B x H)
Wheels (fail-safe)	
Steering wheels (front)	250 x 35 mm
Braking wheels, fixed (rear)	210 x 35 mm
Turn radius	1770 mm
Materials	
Frame	Aluminum (corrosion protection)
Seat	Nylon, black
Seat back	Nylon, black
Tire	Polyurethane
Operating Conditions	
Temperature	0 - 50 °C
Humidity	20 - 80 %
Storage Conditions	
Temperature	-20 bis + 60 °C
Humidity	45 - 60 %

11. REPAIR AND MAINTENANCE FOR THE SPECIALIZED SHOP

It is recommended that the *AT51111* undergo an inspection by the specialized dealer at least once a year. For errors or defects, the *AT51111* is to be immediately handed over to the specialized dealer for repair.

-  Ask the specialized dealer if a replacement device is available, if necessary.
-  Upon request, make necessary information and documents available to the specialized dealer for repair and maintenance.

GENERAL					COMMENTS
The wheeled walker open and folds up easily.					
The wheeled walker goes in a straight line.					
Handles are not loose					
BRAKES					
Tires do not touch during walk					
Brakes are easy to engage					
The brakes have no play					
SEAT					
Seat is not loose					
WHEELS					
No lateral play					
No cracks on the surface					
Back					
Back covering is free of tears					
Back fastening is not loose					

G = inspected inspected by:
 E = replaced R = repair

12. WARRANTY

Model:

Serial number:

Stamp of the point of sale

and legible signature of the Seller:

- 1) ANTAR General Partnership, 03-068 Warsaw, Zawiślańska Street, No. 43 hereby grants a 12 month warranty on the wheelchair from the date of the purchase of the product by the Buyer. For products, the purchase of which was reimbursed by the National Health Fund, the warranty period is extended and equals to half the period of use, specified in the Regulation on guaranteed benefits, provided that the period of use was specified in the Regulation to months or years.
- 2) During the warranty period, ANTAR Company undertakes to make the necessary repairs, in order to restore the proper functioning of the system, within 14 days of receipt of the product by the service, along with the proof of purchase and the warranty card.
- 3) ANTAR reserves the right to replace the product if it considers that the repair costs make it unviable. Repair or replacement of defective products does not extend the warranty period.
- 4) If it is determined that the product functions improperly, it shall be immediately send to the ANTAR's address (shown in point 1) or contact the point of purchase.
- 5) Before operating, please read the instructions and follow them closely. Use of the product contrary to its purpose and recommendations causes irreversible loss of warranty.
- 6) The warranty does not cover damages caused by piercing or cutting the product with the use of a sharp object; damages due to the influence of external mechanical forces; pressure exceeding the maximum permissible weight of the user, as described in the manual; contact the product with alcohol, fatty and oily substances or gasoline. Warranty does not cover parts subject to normal wear during use, such as: tires, soft fabrics, armrests, bearings, rubber elements and others.
- 7) The product being sent to the service as part of the complaint cannot cause epidemics.
- 8) THIS WARRANTY IS VALID ONLY WITH THE PROOF OF PURCHASE (RECEIPT, BILL, INVOICE). PLEASE ATTACH THE PROOF OF PURCHASE IN CASE OF THE SUBMISSION OF ANY CLAIM. FOR RECOGNITION OF EXTENDED PERIOD OF WARRANTY FOR PRODUCTS, WHICH PURCHASE WAS REIMBURSED BY THE NATIONAL HEALTH FUND, PLEASE ATTACH THE COPY OF 'ORDERS FOR MEDICAL SUPPLIES BEING ORTHOPEDIC ITEMS AND AUXILIARIES'.
- 9) Claims submitted without the proof of purchase and the warranty card along with the serial number of the product will not be considered

Manufacturer:

ANTAR Sp. J.

03-068 Warszawa

Ul. Zawiślańska 43

Poland

Tel.: +48 22 518 36 00

Fax: +48 22 518 36 30

