



Adjustable Raised Toilet Seat with Arms

ASSEMBLING INSTRUCTION




SAVE THIS MANUAL FOR FUTURE REFERENCE

Carefully remove contents from the packaging and check all parts against the packing list before discarding packing.

Parts/Hardware List: Please be sure all parts that are listed below are in the carton.

Toilet Seat (Front And Back) 1pc

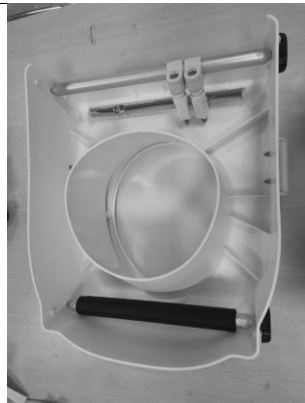

Arm: 2pcs




To Be Continued

Assembling Instruction:

Step 1. Remove existing toilet seat from the bowl.



Step 2. Position the unit onto your toilet bowl, inserting the rear mounting bolts through the existing holes on your toilet.

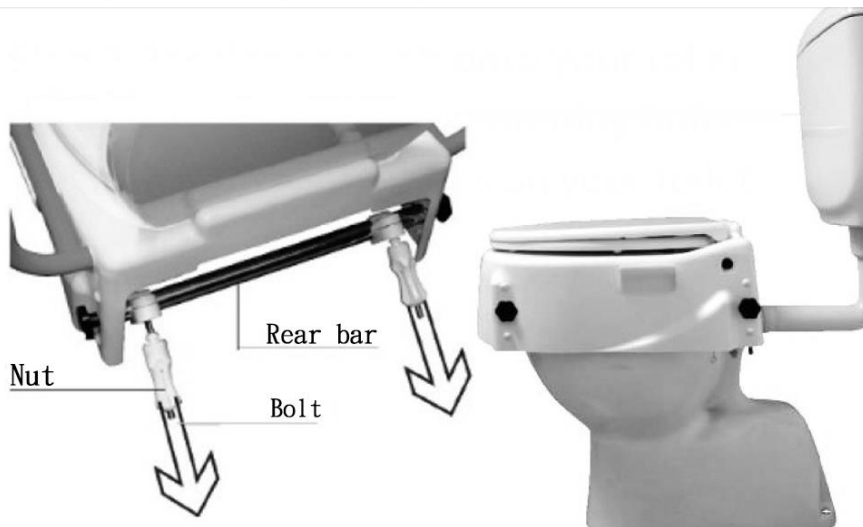

****The position of the mounting bolts can be adjusted to match different toilets. To adjust, loosen & slide the mounting bolts along the rear bar to achieve the correct distance. Reattach nuts and tighten firmly.**

****Ensure toilet seat raiser is firmly clamped against the bowl before use.**

Step 3. Inserting Armrests: The toilet seat raiser can be used with or without armrests.

One or both armrests can be used to suit individual user.


Insert the armrest by pressing the locking button located under the frame of the armrest and slide the armrest into the arm hole at the side of the toilet seat.

**** Ensure secure before use.**



Adjusting seat height:

- The toilet seat raiser can be adjusted to 3 different height settings, adding 2", 4" or 6" to the height of the bowl. Height can be adjusted either before or after installation.
- To adjust the height, undo the front thumb screw bolts, reposition the bar to the desired height and reinsert the thumb screw bolts on both sides and tighten until firm. Adjust the rear bar in the same way. Adjustment can be made to facilitate a forward angled tilt for users requiring extra assistance when standing (rear bar lower than the front bar).
- When adjustment is complete, replace all thumb screw bolt protector caps if the unit is supplied with alternative bolt and cap combinations.

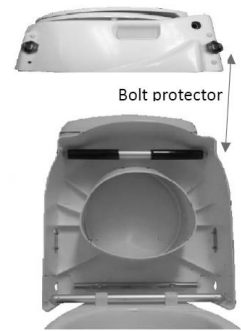

**** Ideal Seat Height**



Ideal height is when the user's feet are flat on the floor, hip height is higher than knee height (ie less than 90 degrees on the knee joint or hip joint)

Care & Maintenance:

- It is very important that you only clean your toilet seat raiser using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners and/or abrasive cleaning pads will seriously damage your product beyond repair.
- Regularly check the mounting screws are tight and that the toilet seat raiser is firmly clamped to the toilet bowl. Do Not use if the seat is cracked or deformed in any way.